
How to Use an MD-Bio Drug Test Strip

Each MD-Bio Lateral Flow Drug Test Strip comes with:

- 1 MD-Bio lateral flow test strip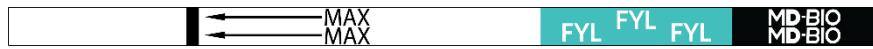

Materials Required but not Included:

- Sterile water
- Clean small container/bottle cap
- Swab/Q-tip

Step 1: Collect a sample of the substance.

1. Fill a cleaned small container with 1 teaspoon of sterile water (or fill a clean bottle cap with sterile water)
2. Take a swab or Q-Tip and wet it by dipping the tip into the sterile water. Unless you are testing a liquid substance, then leave swab tip dry.
3. Swab the substance being tested as defined below:
 - Pills – after wetting the swab tip, directly swab the surface of the pill on both sides. If there are multiple pills within a package they must all be tested prior to use. If one pill is positive or negative, it does not mean they all are.

- Powders – after wetting the swab tip, directly swab a small amount of the substance. Residue on a table or plastic baggy would suffice.

- Syringes – after wetting the swab tip, swab the injection tip of the syringe after the syringe has been loaded with substance.
- Cookers/spoons – after wetting the swab tip, swab a small amount of substance visible on the cooker or spoon.
- Liquids – take the dry swab tip and dunk it into the liquid substance.

Step 2: Insert the collected sample on the swab into the sterile water.

1. After collecting the sample onto the swab, place the swab tip back into the sterile water and stir.
2. Wait for 15 seconds.

Step 3: Insert the MD-Bio lateral flow test strip into the sterile water with the sample mixed in.

1. Remove the MD-Bio lateral flow test strip from its packaging, ensure that you remove the test strip by grabbing the end labeled “MD-Bio”.
2. Place the white end of the test strip into the sterile water. Do not place the strip deeper than the thick black line that says MAX.


3. Wait 10-15 seconds, until a red fluid is visibly flowing up the test strip.

Step 4: Analyze the MD-Bio lateral flow test strip.

1. Remove the test strip from the buffer solution and lay flat.
2. Wait 30-60 seconds.
3. Utilize an MD-Bio analysis chart to analyze the results.
 - One line shows a positive result
 - Two lines shows a negative result

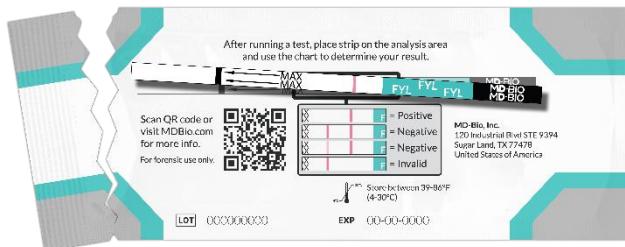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